


Quick Installation Guide

We would like to thank you for the trust you have placed in us in purchasing this wheelchair.

Attaching the drive wheels

The wheels can be attached and removed without tools.

Press the locking button (1) on the wheel to attach the wheel to the frame. Release the locking button to fix the axle in place. Ensure that the locking button springs back into its initial position and the wheel is firmly attached to the frame.


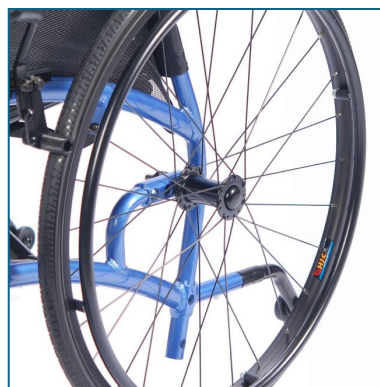


Every time you refit the drive wheels, check to make sure that the axle is securely fitted back in place by trying to remove the wheel without pressing the locking button. The wheel should not come away from the frame.

Attaching the anti-tip wheels

Anti-tip wheels stop the wheelchair from tipping backwards. They must be attached before use. Press the locking button and push the wheel into the frame with the anti-tip wheel showing downwards (2).

If the wheelchair is to go up or down a high step (e.g. a curb), the anti-tip wheels must be swiveled upwards to avoid them catching on the ground (3). Pull on them to unlock and swivel up/down.

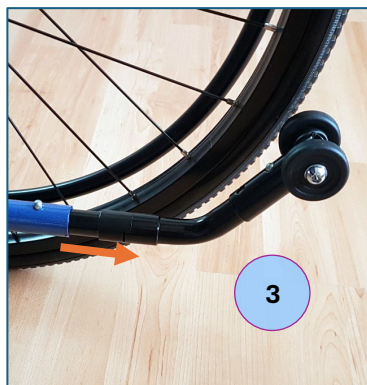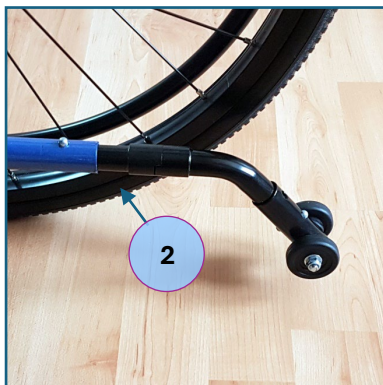

The anti-tip wheels must be secured in their apertures and swiveled downwards when the wheelchair is in use (2) (exception: when going over a high step). Otherwise, the wheelchair could tip over and cause serious injuries. This must be checked after each adjustment and especially when the anti-tip wheels have been temporarily removed and then refitted.